DATI DEL FABBRICANTE
Nome ditta: Pasini Pier Maurizio S.r.l.
Sede ditta: via Flavio Torti, 5 – 15053 – Castelnuovo Scrvia (AL)
P.IVA: 02223360062
PEC: pasinipiermauriziosrl@pec.it
Tel. Ufficio: +39-0131826382
E-mail: piermauriziopasini@gmail.com
Site: www.hollythemask.com

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PRODUCER DETAILS:
Company name: Pasini Pier Maurizio S.r.l.
Company headquarter: Via Flavio Torti, 5 – 15053 – Castelnuovo Scrvia (AL) - ITALY
VAT Nr. IT02223360062
PEC: pasinipiermauriziosrl@pec.it
Office telephone number: +39 0131 826382
E-mail: piermauriziopasini@gmail.com
Website: www.hollythemask.com
MEDICAL DEVICE REGISTERED WITH THE ITALIAN MINISTRY OF HEALTH (BD/RDM N°1966037)

AUTHORISED SERVICE:
Assistance and maintenance work should only be performed by a service technician authorised and qualified by the company Pasini Pier Maurizio S.r.l.

PRESENTATION OF THE MANUAL:
The following manual contains instructions on how to use the masks with replaceable filters. This manual is divided into different sections, each one regarding a series of topics, subdivided in chapters and paragraphs. The general index lists all the topics of covered by the user manual. The numbering of the pages is progressive and the page number is displayed at the bottom of each page. This manual is intended for the user responsible for the use and storage of the medical device and is related to the technical life after its production and sale.
This manual reserved property information and shall not be shared or supplied to third parties in any form, without any manufacturer's prior written consent.
Pasini Pier Maurizio S.r.l. declares that the information contained in this manual is congruent with the technical and safety specifications of the medical device to which the manual refers.
A certified copy of this manual is stored in the device technical file, preserved at Pasini Pier Maurizio S.r.l..
Pasini Pier Maurizio S.r.l. does not recognise any documentation that has not been issued, released or distributed by itself or any authorised representative.
This manual, as for the technical file, will be kept by the manufacturer throughout the period required by law. During this period, a copy of the documentation accompanying the product may be provided upon request at the time of sale.
The entire technical file remains available for this period to the supervisory authorities exclusively, who may request a copy. After this period, it will be responsibility of whoever manages the product to ensure that both product and documentation comply with the laws in force, in order to be able to use the masks accordingly.

LOT	SERIAL NUMBER	CE	CE MARKING
	PRODUCTION DATE		PROTECT THIS MD FROM ATMOSPHERIC AGENTS
	PRODUCER		PROTECT THIS MD FROM DIRECT SUNLIGHT
	READ USER MANUAL BEFORE USING MASK		RECYCLABLE PACKAGING

WARRANTY
The warranty rules, listed in full in the purchase contract, are valid only if the medical device is used under the conditions of intended use. Any repair or modification made to the device by the user or by unauthorised companies will void the warranty. The warranty does not extend to damage caused by inexperience or negligence in the use of the device.

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The products sold are covered by a warranty regarding the following terms and conditions:
1. This warranty is valid for a period of twelve (12) or twenty-four (24) months depending on the legal nature of purchaser.
2. The manufacturer undertakes to replace incorrectly manufactured products, at its own discretion, only after accurate product control procedures and verification of bad construction.
3. Transport and/or shipping costs are always borne by purchaser in case of incorrect use of the warranty terms.
4. During the warranty period, all replaced components become property of the manufacturer.
5. Only the original purchaser who has respected the instructions for normal use outlined in the user manual can benefit from this warranty. The warranty liability expires when: the device is transferred from the original owner to another party, or changes have been made to it.
6. The warranty does not include damage resulting from excessive solicitation such as the use of the device after any anomaly detections, the use of inadequate operating methods as well as the failure to observe indications listed in the user manual.
7. The manufacturer assumes no responsibility for any difficulties that may arise in the resale or use abroad due to foreign legal provisions.
Notice: if the use of the present warranty is deemed necessary, please provide us with the following data:
1. Mask type
2. Purchase date (presentation of the purchase document)
3. Detailed problem description

MEDICAL DEVICE DESCRIPTION
The medical devices described in this user manual are washable and sterilisable masks with replaceable filters, intended for use by medical and nursing staff in order to protect patients during operations without exposing them to any risk. The mask is a mechanical barrier against materials and substances.
The mask is in fact used to avoid contamination towards the patient during medical activity.
The mask is unisex and available in different sizes: Holly by Pasini for adults, Holly Small and Bunny by Holly for children from 6 to 10 years.
The medical device is not sterile and is reusable.

TECHNICAL DATA
Materiali:
• Main structure: 100% E.V.A. expanded (soft and comfortable, hypoallergenic, non-toxic, waterproof and resistant to bacteria and fungi)
• NW filter 100% polypropylene - Non-woven fabric, triple layer (water-repellent, waterproof but breathable, non-toxic, hypoallergenic, soft to the touch and non-abrasive)
• Elastic bands h 7 mm, latex composition covered with polyester yarn
• Brass snap buttons diameter 12 mm
Size: Holly by Pasini - **Dimensions (mm):** 180 x 60 x 75 — **Size:** Holly Small - **Dimensions (mm):** 155 x 60 x 50 — **Size:** Bunny by Holly - **Dimensions (mm):** 140 x 50 x 50
Type in accordance with standard UNI EN 14683:2019: Type II - **Bacterial filtration efficiency (BFE), (%)**: ≥ 98 - **Differential pressure (Pa/cm2):** 16,9 - **Microbial cleaning (ufc/g):** ≤ 30
Field of application and intended use
The medical device shall be used:
• to avoid contamination of the patient
Unintended use
No other use than that described in the paragraph "FIELD OF APPLICATION AND INTENDED USE" is expected. It is also absolutely forbidden:
• To use the mask as an individual protective device (DPI).
• To use the medical device before or without reading the user manual carefully.
• To be used by people allergic to any of the materials the mask is produced with.
• To be used by children.
• To use the device for applications that differ from those indicated in the "FIELD OF APPLICATION FOR INTENDED USE".
• To apply the mask on injured skin or on a wound.
• To apply the medical device on parts of the body other than the face.

TECHNICAL DATA
Materiali:
• Main structure: 100% E.V.A. espansa (soft and comfortable, hypoallergenic, non-toxic, waterproof and resistant to bacteria and fungi)
• NW filter 100% polypropylene - Non-woven fabric, triple layer (water-repellent, waterproof but breathable, non-toxic, hypoallergenic, soft to the touch and non-abrasive)
• Elastic bands h 7 mm, latex composition covered with polyester yarn
• Brass snap buttons diameter 12 mm
Size: Holly by Pasini - **Dimensions (mm):** 180 x 60 x 75 — **Size:** Holly Small - **Dimensions (mm):** 155 x 60 x 50 — **Size:** Bunny by Holly - **Dimensions (mm):** 140 x 50 x 50
Type in accordance with standard UNI EN 14683:2019: Type II - **Bacterial filtration efficiency (BFE), (%)**: ≥ 98 - **Differential pressure (Pa/cm2):** 16,9 - **Microbial cleaning (ufc/g):** ≤ 30
Field of application and intended use
The medical device shall be used:
• to avoid contamination of the patient
Unintended use
No other use than that described in the paragraph "FIELD OF APPLICATION AND INTENDED USE" is expected. It is also absolutely forbidden:
• To use the mask as an individual protective device (DPI).
• To use the medical device before or without reading the user manual carefully.
• To be used by people allergic to any of the materials the mask is produced with.
• To be used by children.
• To use the device for applications that differ from those indicated in the "FIELD OF APPLICATION FOR INTENDED USE".
• To apply the mask on injured skin or on a wound.
• To apply the medical device on parts of the body other than the face.

The products sold are covered by a warranty regarding the following terms and conditions:
1. This warranty is valid for a period of twelve (12) or twenty-four (24) months depending on the legal nature of purchaser.
2. The manufacturer undertakes to replace incorrectly manufactured products, at its own discretion, only after accurate product control procedures and verification of bad construction.
3. Transport and/or shipping costs are always borne by purchaser in case of incorrect use of the warranty terms.
4. During the warranty period, all replaced components become property of the manufacturer.
5. Only the original purchaser who has respected the instructions for normal use outlined in the user manual can benefit from this warranty. The warranty liability expires when: the device is transferred from the original owner to another party, or changes have been made to it.
6. The warranty does not include damage resulting from excessive solicitation such as the use of the device after any anomaly detections, the use of inadequate operating methods as well as the failure to observe indications listed in the user manual.
7. The manufacturer assumes no responsibility for any difficulties that may arise in the resale or use abroad due to foreign legal provisions.
Notice: if the use of the present warranty is deemed necessary, please provide us with the following data:
1. Mask type
2. Purchase date (presentation of the purchase document)
3. Detailed problem description

MEDICAL DEVICE DESCRIPTION
The medical devices described in this user manual are washable and sterilisable masks with replaceable filters, intended for use by medical and nursing staff in order to protect patients during operations without exposing them to any risk. The mask is a mechanical barrier against materials and substances.
The mask is in fact used to avoid contamination towards the patient during medical activity.
The mask is unisex and available in different sizes: Holly by Pasini for adults, Holly Small and Bunny by Holly for children from 6 to 10 years.
The medical device is not sterile and is reusable.

TECHNICAL DATA
Materiali:
• Main structure: 100% E.V.A. expanded (soft and comfortable, hypoallergenic, non-toxic, waterproof and resistant to bacteria and fungi)
• NW filter 100% polypropylene - Non-woven fabric, triple layer (water-repellent, waterproof but breathable, non-toxic, hypoallergenic, soft to the touch and non-abrasive)
• Elastic bands h 7 mm, latex composition covered with polyester yarn
• Brass snap buttons diameter 12 mm
Size: Holly by Pasini - **Dimensions (mm):** 180 x 60 x 75 — **Size:** Holly Small - **Dimensions (mm):** 155 x 60 x 50 — **Size:** Bunny by Holly - **Dimensions (mm):** 140 x 50 x 50
Type in accordance with standard UNI EN 14683:2019: Type II - **Bacterial filtration efficiency (BFE), (%)**: ≥ 98 - **Differential pressure (Pa / cm2):** 16,9 - **Microbial cleaning (ufc / g):** ≤ 30
Application area and intended use:
The medical device shall be used:
• to avoid contamination of the patient
Unintended use
No other use than that described in the paragraph "FIELD OF APPLICATION AND INTENDED USE" is expected. It is also absolutely forbidden:
• To use the mask as an individual protective device (DPI).
• To use the medical device before or without reading the user manual carefully.
• To be used by people allergic to any of the materials the mask is produced with.
• To be used by children.
• To use the device for applications that differ from those indicated in the "FIELD OF APPLICATION FOR INTENDED USE".
• To apply the mask on injured skin or on a wound.
• To apply the medical device on parts of the body other than the face.

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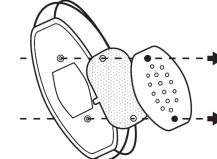
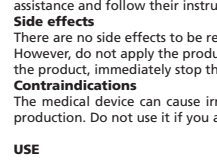
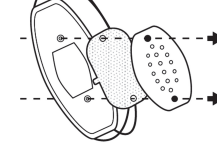
Limits of the medical device:
- Masks do not cure any disease.
RESIDUAL RISKS
• It is absolutely forbidden to apply any modification to the medical device. Any damage resulting from the use of the device, improperly modified by an unauthorised operator, relieves the manufacturer of any responsibility.
• Save this user manual for future reference. It is necessary for a correct and safe use of the medical device.
• If the structure of the device shows deformations after an accidental impact, which could be a friction against an abrasive material, such as to make it or assume it dangerous and/or unusable, it is necessary to contact authorised assistance and follow their instructions.
Side effects
There are no side effects to be reported in the user manual.
However, do not apply the product on your skin when there is a wound or irritation. In case of irritation caused by the product, immediately stop the application and consult a doctor.
Contraindications
The medical device can cause irritation on subjects particularly sensitive or allergic to the materials used for its production. Do not use it if you are allergic to one of the materials indicated in the paragraph "Technical data".
USE
Use
• Verify that the mask is clean before application. If not, follow the operations indicated in paragraph "Cleaning and maintenance".
• Sanitise your hands by washing them thoroughly with soap and water or with a hydroalcoholic disinfectant solution.
• Remove the mask from the packaging (in case of first use).
• Put on the mask positioning the elastic bands one at a time behind the head.
• Place the mask on the face and cover nose, mouth and chin.
How to replace the filter:
• Sanitise your hands by washing them thoroughly with soap and water or with a hydroalcoholic disinfectant solution.
• Remove the front part of the mask by opening the snap buttons located on the sides of the structure.
• Remove the used filter from its seat.
• Disinfect the mask.
• Apply the new filter by inserting its holes in correspondence with the snaps.
• Close the front of the mask by pressing the snap buttons.
Dispose of the filter removed by referring to local regulations of unsorted municipal waste.

CLEANING AND MAINTENANCE
• **Washing:** remove the filter, gently rub the cover of the mask with a cloth moistened in warm water and with neutral and non-aggressive detergents. We recommend sanitising the product immediately after purchase.
• **Disinfection:** follow the instructions of the disinfectant system supplier.
• **Spray** on the surface with disinfectant solutions diluted with hot water and wait for drying.
Storage
Before storing the medical device, make sure it is clean.
• Store the medical device in a cool indoor environment, away from atmospheric agents and steam.
• Store the medical device away from heat sources, open flames and direct sunlight.
• After use, place the washed and completely dried mask inside a case, protected from direct sunlight, from atmospheric agents, heat sources, dust, liquids and humidity.
Disposal
When finished using the medical device, proceed with its disposal.
The construction materials of the medical device do not require special disposal procedures. It is necessary to refer to local regulations for the disposal of unsorted waste.
Do not disperse or abandon the product to the environment for any reason

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Holly By Pasini, Holly Small, Bunny By Holly

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